

GE0012 Fuel Cell Alcohol Breath Tester

Quick Installation Guide

1. Please install 3 x AAA Batteries. Please ensure that batteries are fresh and do not mix different types of batteries (alkaline and rechargeable types)
2. Install the Plastic Mouth Piece
3. Turn the Alcohol Breath Tester on by pressing the On Button on the front
4. To proceed with testing, press the on button quickly once.
5. This will initiate the sensor to Pre-heat.
6. The LCD Screen will flash PH for a few seconds
7. The Screen will then indicate Test on the screen - At this stage, the unit will start beeping
8. Blow into the mouth at a slow and steady pace - At this stage, the beep will cease and the unit will sound continuously during breath sampling
9. After blowing for about 10 seconds, the unit will vibrate and cease all sounds to indicate a successful sampling
10. If the sampling is unsuccessful, it will indicate error. Please follow the steps again and ensure that air is blown into the mouth piece at a steady pace for 10 seconds.
11. If successful, the LCD will indicate your reading in %BAC.
12. To turn off the Alcohol Breath Tester, press the on button for 3 seconds.

Note

- Do not blow too hard on the mouth piece. This will produce an error prompt.
- This device is for indicative purposes only and may only be used as a guide.